
How to reinstall or upgrade LB120


A. Double click "This PC" from desktop

B. Press “Uninstall or change program”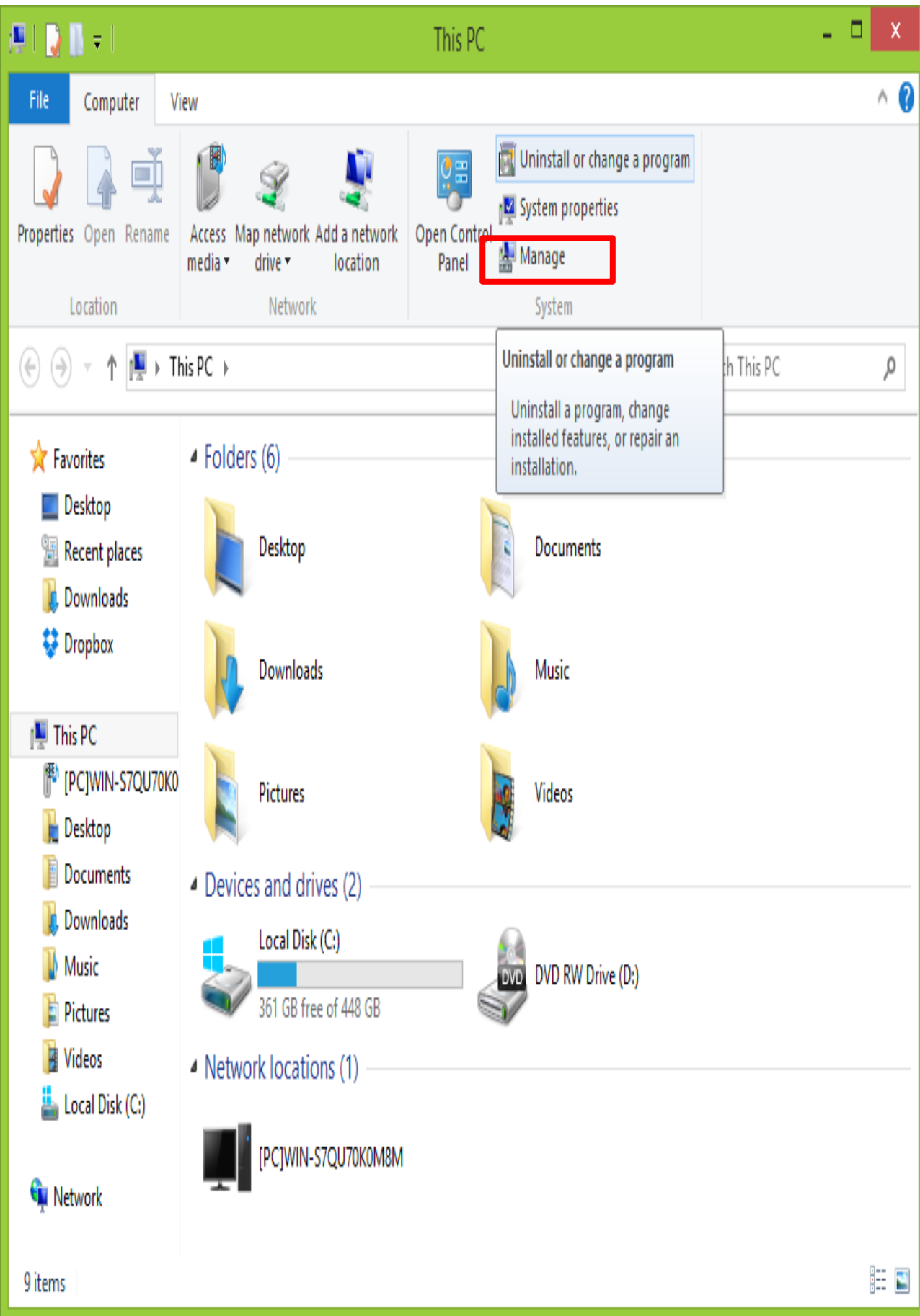

C. Select the program and press “Uninstall”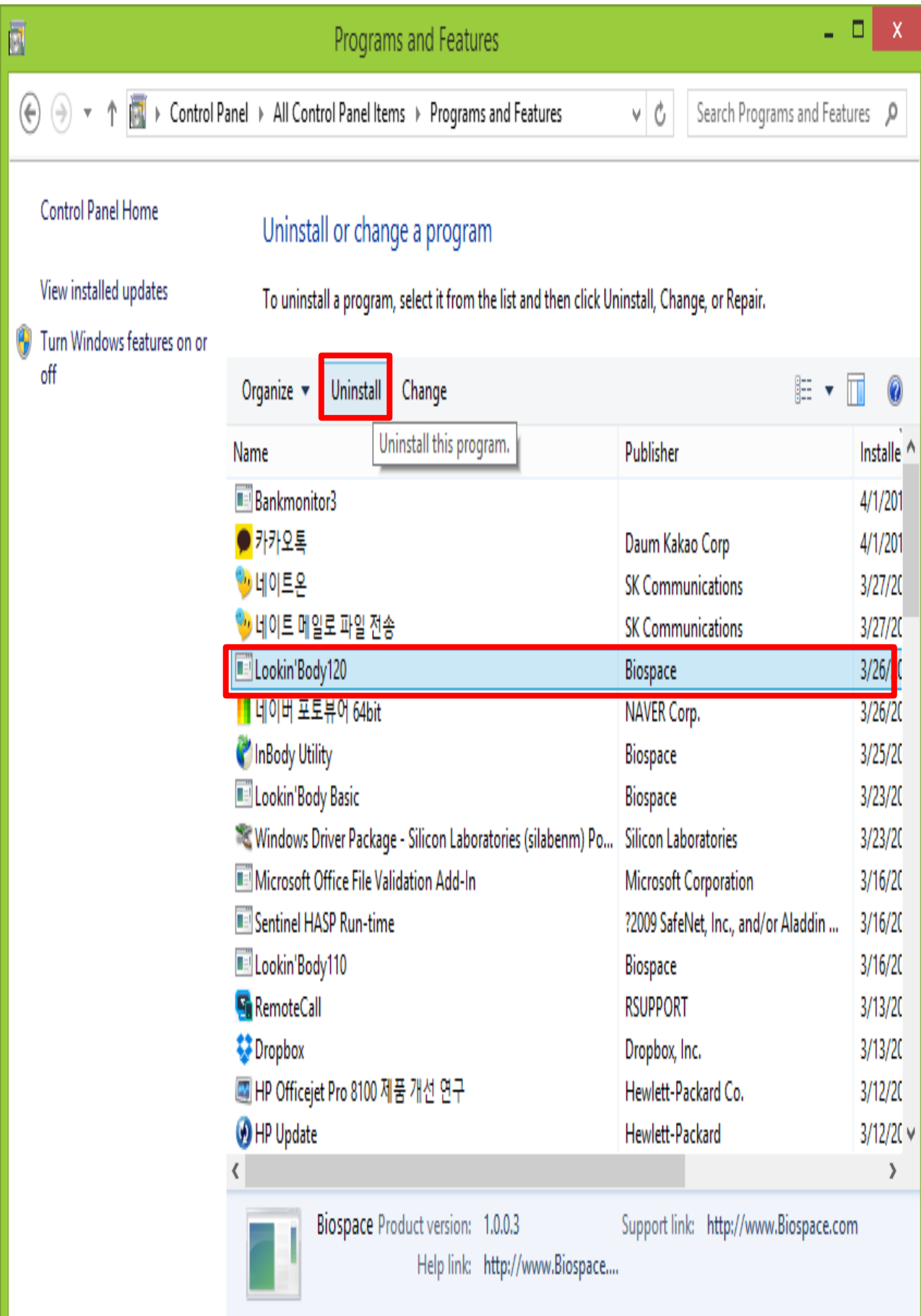

Control Panel Home

Uninstall or change a program

To uninstall a program, select it from the list and then click Uninstall, Change, or Repair.

View installed updates

Turn Windows features on or off

Organize ▾ **Uninstall** Change

Name	Publisher	Installed
Bankmonitor3		4/1/2011
카카오톡	Daum Kakao Corp	4/1/2011
네이트온	SK Communications	3/27/2011
네이트 메일로 파일 전송	SK Communications	3/27/2011
Lookin' Body120	Biospace	3/26/2011
네이버 포토뷰어 64bit	NAVER Corp.	3/26/2011
InBody Utility	Biospace	3/25/2011
Lookin' Body Basic	Biospace	3/23/2011
Windows Driver Package - Silicon Laboratories (silabnm) Po...	Silicon Laboratories	3/23/2011
Microsoft Office File Validation Add-In	Microsoft Corporation	3/16/2011
Sentinel HASP Run-time	?2009 SafeNet, Inc., and/or Aladdin ...	3/16/2011
Lookin' Body110	Biospace	3/16/2011
RemoteCall	RSUPPORT	3/13/2011
Dropbox	Dropbox, Inc.	3/13/2011
HP Officejet Pro 8100 제품 개선 연구	Hewlett-Packard Co.	3/12/2011
HP Update	Hewlett-Packard	3/12/2011

Lookin' Body120 Product version: 1.0.0.3 Support link: <http://www.Biospace.com>
Help link: <http://www.Biospace.com>

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
InstallShield

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E. -Double Click "Local Disk(C:)"

The screenshot shows the Windows File Explorer interface for 'This PC'. The ribbon at the top includes 'File', 'Computer', 'View', and 'Manage' tabs. The ribbon contains icons for 'Properties', 'Open', 'Rename', 'Access media', 'Map network drive', 'Add a network location', 'Open Control Panel', 'Uninstall or change a program', 'System properties', and 'Manage'. The address bar shows 'This PC' and a search box labeled 'Search This PC'. The main area is divided into 'Folders (6)', 'Devices and drives (2)', and 'Network locations (1)'. The 'Local Disk (C:)' is highlighted with a red box, showing a progress bar and the text '361 GB free of 448 GB'. The 'DVD RW Drive (D:)' is also visible. The left sidebar shows 'Favorites' (Desktop, Recent places, Downloads, Dropbox) and 'This PC' (PC)WIN-S7QU70K0, Desktop, Documents, Downloads, Music, Pictures, Videos, Local Disk (C:), and Network. The bottom status bar indicates '9 items' and '1 item selected'.

F. Right Click the designated folder and press "Delete"

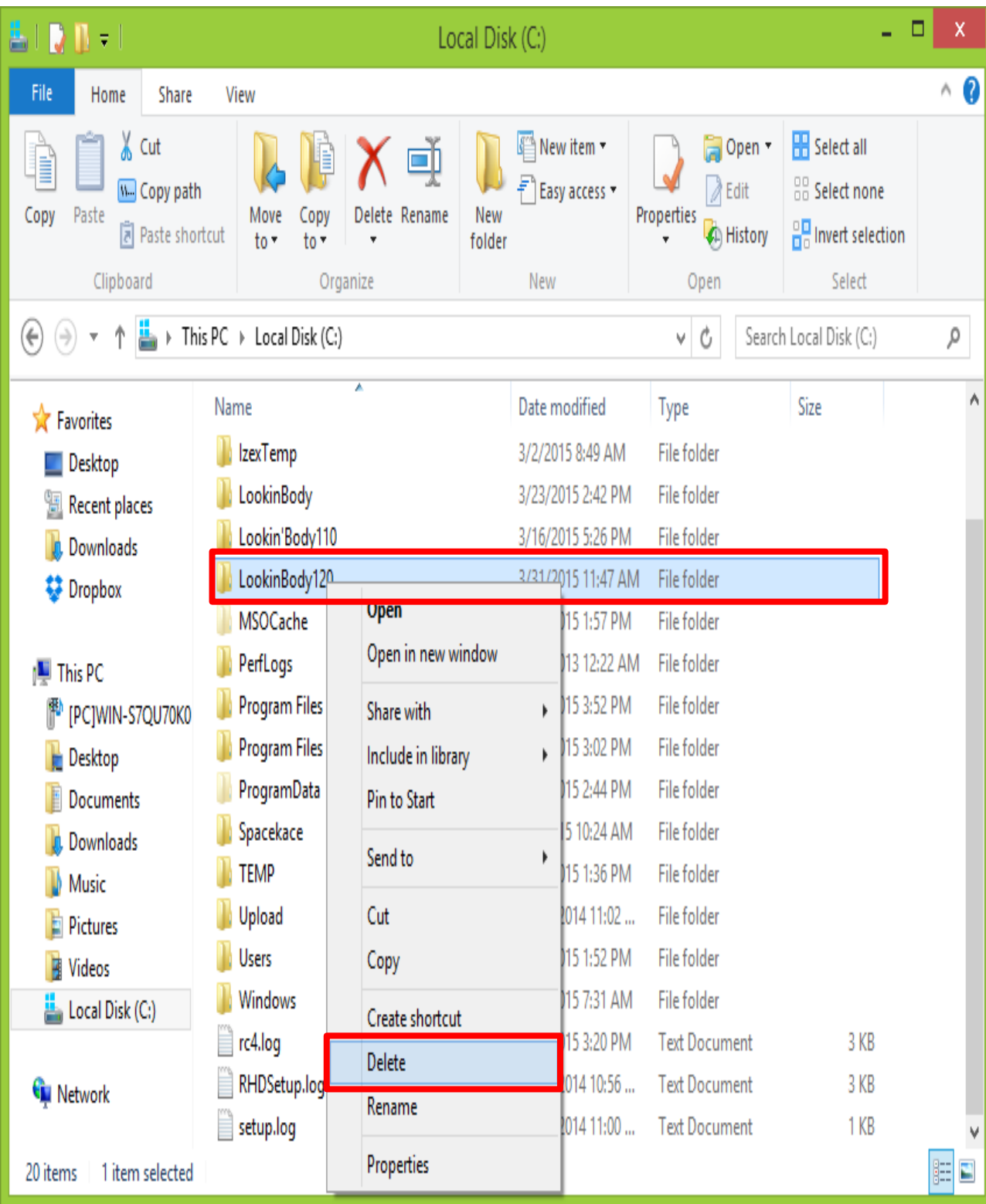**Precaution: DB file will be also erased.
Please backup the DB**

G. Connect to Dropbox and download LB120

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<https://www.dropbox.com/home/InBodyCS>

H. Select "LB120" to find the "Installation Folder"

Dropbox > InBodyCS > Lookin'Body

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.NET Framework	폴더	--
----------------	----	----

Accessory	폴더	--
-----------	----	----

Drivers	폴더	--
---------	----	----

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LB Basic	폴더	--
----------	----	----

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I. Select "Installation File" to find the installation file

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InBody CS Department ▾

> Lookin'Body > LB 120

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[Installation Files]

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--

Bluetooth Connection with IB570

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--

Complete Uninstallation and Reinstallation

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--

Custom Logo Input

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--

DB Backup Importation from IB570

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--

Previous LB Data Importation

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--

Wi-Fi Setup

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--

Bluetooth Dongle (For InBody vs For PC).png

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K. There will be a zip file on your Desktop

This PC

LookinBod...

L. Unzip the file and you a new folder will appear

M. Open the folder and start the installation

The screenshot shows a Windows Explorer window titled "Lookin'Body120_Setup". The ribbon includes "File", "Home", "Share", and "View". The ribbon tabs are "Clipboard", "Organize", "New", "Open", and "Select". The address bar shows the path "Lookin'Body120_Setup" and a search box. The left sidebar shows "Favorites" (Desktop, Recent places, Downloads, Dropbox) and "This PC" (This PC, [PC]WIN-S7QU70K0, Desktop, Documents, Downloads, Music, Pictures, Videos, Local Disk (C:), Network). The main pane displays a list of files and folders:

Name	Date modified	Type	Size
ISSetupPrerequisites	3/26/2015 1:29 PM	File folder	
Ox0409.ini	3/23/2010 4:44 PM	Configuration sett...	22 KB
Ox0411.ini	3/16/2012 12:55 PM	Configuration sett...	15 KB
Ox0412.ini	4/3/2012 2:44 PM	Configuration sett...	14 KB
Autorun.inf	9/30/2013 2:12 PM	Setup Information	1 KB
data1.cab	3/26/2015 10:48 AM	CAB File	1,034 KB
data1.hdr	3/26/2015 10:48 AM	HDR File	285 KB
data2.cab	3/26/2015 10:48 AM	CAB File	459,482 KB
ISSetup.dll	5/13/2014 9:10 AM	Application extens...	590 KB
layout.bin	3/26/2015 10:48 AM	BIN File	3 KB
Lookin'Body120_Setup.exe	3/17/2015 3:08 PM	Application	831 KB
setup.ini		File description: InstallScript Setup Launcher Unicode t...	4 KB
setup.inx		Company: Biospace	237 KB
setup.isn		File version: 1.0.0.3	55 KB
User's_Manual_eng.pdf		Date created: 3/26/2015 1:29 PM	7,091 KB
User's_Manual_kor.pdf	3/12/2014 3:29 PM	Adobe Acrobat D...	7,680 KB

The file "Lookin'Body120_Setup.exe" is highlighted with a red box. A tooltip is displayed over it, showing the following properties:

- File description: InstallScript Setup Launcher Unicode t...
- Company: Biospace
- File version: 1.0.0.3
- Date created: 3/26/2015 1:29 PM
- Size: 831 KB

At the bottom left, it says "16 items".

End of instructions.

Go back to Technician's troubleshooting steps.