
# nBody720 Quick Installation Guide

### 1. Contents of the Box

When opening the box, make sure all the following items are inside.

#### A. Included Items

- InBody720 unit
- Quick installation guide
- User's manual CD
- Power cord (AC 125V, 10A, 1.8m) (1EA)
- 6 Hinge cover + Hexagonal wrench (6mm), 1 each
- 1 Fuse holder + 2 Fuses(F2.5AH 250V) + 2 spare fuses
- Results Interpretation Guide (1EA)
- InBody720 Poster (Optional)
- Results Sheet Box (500 Sheets /1EA, 2Box)(Optional)
- Results Interpretation Poster (1EA)(Optional)

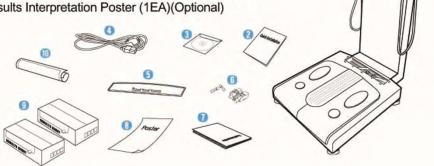

### 2. Installation Instructions


#### A. Workplace Requirements

- (1) Indoor installation is highly recommended. Any outdoor area where the equipment is to be located should meet all the conditions below.
- (2) Operation environment: 10 ℃~40 ℃ (50 °F~104 °F), 30%~75% RH, 70~106kPa

#### B. Attention


Please READ the following CAREFULLY before installing the InBody720.

- (1) The InBody720 should be placed horizontal to the ground.
- (2) The level indicator is located at the center of connecting part and leveling screws under the lower part. Check the level indicator.

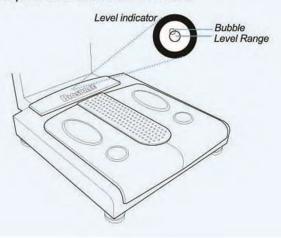




Unpack the carton and remove the @, b pads.

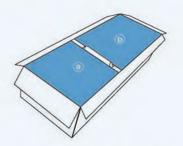


Straighten the equipment as pulling it forward at the same time.

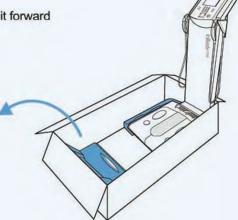




1 Tighten the screws connecting upper and lower part with a hexagonal wrench.

The hexagonal wrench is located underneath the hinge cover.

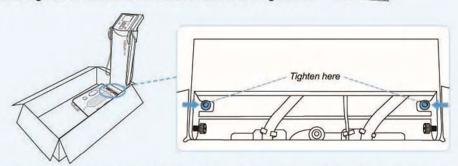


Mold the upper part and center of lower part with two people and take the equipment out of the carton. Place the equipment at desired area and tighten the 4 screws connecting upper and lower parts with a hexagonal wrench.

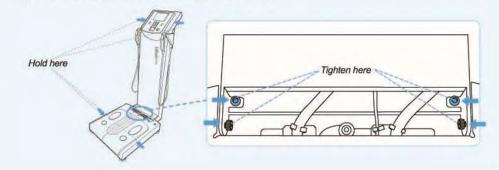

## 5. Level InBody720

The level indicator is located at the center of connecting part and leveling screws under the lower part. Check the level indicator. You may adjust 5 leveling screws under the lower part to level the equipment.

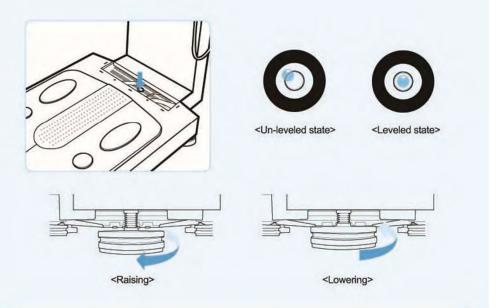


4

Be careful when handling screws, your fingers may get hurt on the edge of the base frame

# 6. Close the hinge cover


Attach the hexagonal wrench back to the underneath of hinge cover.

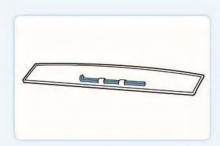

OClose the hinge cover.



SLift up the rear part and remove the elastic bands pulling the hand electrode cables.

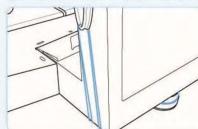

# InBody720 Repacking

Be sure to turn off the power switch and unplug the power cable before repacking. Be careful not to damage foot and hand electrodes while repacking.

- Turn off the power and remove the power cable.
- Use elastic band to hold hand electrode cables. Lift up the rear and slightly, hook the elastic band to the rear level screw, pass through the cable and hook it again to the rear level screw.
- Solution of a carton and put InBody720 on the top.
- Place the head pad on a carton.
- Open the hinge cover and loosen the 4 screws.
- (1) Fold the upper part and put in the carton.
- Insert the accessory bag in the carton.
- Put upper pads on the top.
- Olose the carton box and seal it with tape.

