

LB120: Data Transfer From InBody

1. Insert your InBody USB thumb drive into the back USP port of your InBody



2. Once you have inserted the USB thumb drive, go to option 07. Export Data as Excel. Choose select all and then press START . Once the export has been completed, please remove the USB thumb drive and insert it into your computer



3. Once you have inserted the InBody USB thumb drive. Open up the LB120 and perss the Setup on the top left corner



4. Go to Option 18. DATA RESTORATION/ IMPORTATION

up	
01. Results Sheet Types/Paper Types/Printing Options/Printer/Automatic Printing Options	
02. Outputs/Interpretations for Results Sheet	
03. Results Sheet Custom Logo	
04. Reference Range	
05. E-mail Options	
06. InBody Model/Country/Language/Units/Date Format/Password	
07. N/A	
08. N/A	
09. Data Sharing	
10. Export Data as CSV/Image Files	
11. Customer Service Information	
12. Auto-Lock	
13. Program and Computer Information/Update History	
14. Export Data as Excel	
15. Import Group Registration Data as Excel	
16. Reintstallation Guide	
17. Data Backup	
18. Data Restoration/Importation	
19. Temporary Data	
20. Import Data from Previous Lookin'Body	

5. Once you are in Option 18, please press the button on the bottom and select the file that was unzipped earlier and choose the lookinbody folder.

o. Buta Restoration importat	ion	×
Data Restoration will restore m Data Importation will import sav	ember information, test results, and user settings from a previous Lookin'Body backup file. red data from the InBody by using a USB Thumb Drive.	
1. Restore data from a previous	s Lookin'Body backup file	
1) Click [Data Restoration/Impo	ortation] below.	
2) Select the folder that has the	e backup file.	
2 Import data caved from the l	Redu by using the USB Thumb Drive	
1) Click [Data Restoration/Impr		
2) Select the LISB Thumh Drive	a used to export data from the InBody. Click the "inbody" folder, then select the "lookinbody" folder	
2) ociect the oob many birt		
	Click the button below to restore data.	
	Data Restoration/Importation	
	Browse For Folder	
	Select the folder where Lookin'Body data is backed up, then	
	Click [OK] to restore.	
	▷ p∰ Computer ^	
	Network	
	V V V V V V V V V V V V V V V V V V V	
	 Network Control Panel Recycle Bin Inbody lookinbody 	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	
	Ver Network Control Panel Recycle Bin Disokinbody Lookinbody E	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	
	Network Image: Control Panel Image: Recycle Bin Image:	
	 Vework Control Panel Recycle Bin Inbody Iookinbody 	
	 Network Control Panel Recycle Bin Inbody Iookinbody 	

6. Once the transfer has been completed, you will receive a confirmation message and the it will prompt that the software program will restart. Once the LB120 re-opens, press LIST ALL to verify that restoration was successful.

18. Data Restoration/Importation				×
Data Restoration will restore membe Data Importation will import saved da	r informat Ita from th	ion, test results, and user settings from a p e InBody by using a USB Thumb Drive.	revious Lookin'Body backup file.	
1. Restore data from a previous Lool	kin'Body b	packup file		
1) Click [Data Restoration/Importation	al below			
2) Select the folder that has the bac	Lookin'	Body 🛛		
 2. Import data saved from the InBod 1) Click [Data Restoration/Importatio 2) Select the USB Thumb Drive use 	1	Lookin'Body will restart to apply changes.	select the 'lookinbody' folder.	
		ΟΚ		